

## **DECONTAMINATION POLICY**

All manikins will be decontaminated immediately after the program is completed following the manufacturer's suggestions for cleaning.

- All expendable parts will be disposed of immediately.
- Faces must be scrubbed with warm, soapy water and rinsed thoroughly. The faces should then be soaked in a chlorine solution (1/4 cup chlorine to 1 gallon of water) for a minimum of 10 minutes and then rinsed thoroughly and dried. The chlorine solution should always be made fresh for each class and disposed of after each cleaning session.
- Chests should be wiped clean using some type of cleaning solution.

During a training session, manikins should be cleaned between each student by vigorously wiping the inside and outside of the manikin's face with 70% alcohol. Wipe with alcohol and let air-dry at least 30 seconds. After that time any remaining solution can be wiped dry.